



## *Comfort Fit*

### Wheelchair Back Cushion



#### Features

- Back tension is adjustable to fit specific users and wheelchair width.
- Pressure reduction foam provides a more comfortable lumbar support.
- Design allows for the backrest to fold with the wheelchair.



#### Easy Installation

- Remove existing wheelchair back.
- Slide Comfort-Fit back cushion directly over wheelchair canes and secure.
  - Adjust tension straps to fit specific user.
  - Add foam lumbar cushion to tension straps.



Model # CF-1620 (16"-20" wide chair) HCPCS: E2611  
Model # CF-2230 (22"-30" wide chair) HCPCS: E2612

Manufactured by Telemade, 3215 Huffman Eastgate, Huffman, TX 77336



*Specialty Medical Products*  
*Made in the U.S.A.*  
**(281) 324-2900**